

Correction of Upper Extremity Contractures

MiniRail System

Part C: M2 MultiPlanar MiniRail

By Dr. L. Andrew Koman

| | Page N° |
|---|---------|
| GENERAL INFORMATION | 1 |
| LONGITUDINAL DEFICIENCY OF THE RADIUS (RADIAL CLUB HAND) | 2 |
| Soft Tissue Correction Prior to Centralization or Radialization | 2 |
| Post Operative Management | 4 |
| CORRECTION OF WRIST CONTRACTURE SECONDARY TO SPASTICITY | 6 |
| Post Operative Management | 9 |

GENERAL INFORMATION

The M2 is provided as a non-sterile device (see page 11 for cleaning and sterilization instructions).

There are a variety of screws available for this device, as listed on the back cover of this guide.

Note: In this technique we use the term screw instead of wire. The screws have 3 mm diameter shafts with different sized threaded tips and the wires have 2 mm diameter shafts (see table on back page). The threaded wires are more applicable for use in the Radial Club Hand procedures, as the majority of patients are age three years or less and have very small bones.


The largest selected diameter screw should be compatible with the patient's bone diameter. The hole in the bone should not be larger than 30% of the bone diameter, and should be in the center of the bone axis. This is particularly important for stability when lengthening.

All implants are self-drilling. The wires can be inserted directly percutaneously; the screws should be inserted through a small incision.

Note: Because they have different shaft diameters, screws and wires should NEVER be positioned in the same clamp.

All the screws have tapered threads. Care should be taken not to insert them too far. They should **not** be backed out, as loosening will occur.

The 1.6 and 2.0 mm wires have cylindrical threads, and can be backed out if necessary.

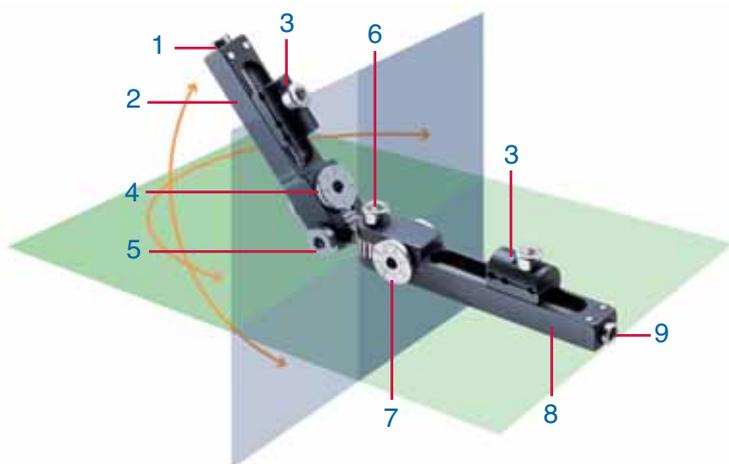

In the diaphysis, screw or wire insertion should be in the center of the bone axis, to avoid weakening the bone.

Note: If 2.5 -2.0 mm threaded screws are used and require to be backed out; a screw with a larger diameter thread can be inserted.

The MultiPlanar Geared MiniRail M2 permits staged correction of complex joint deformities and nonsurgical lengthening of musculotendinous contractures. The device consists of 2 linked monorails connected by paired gears. It is constructed to allow positioning of radio-ulnar or dorso-palmar gears over the center of rotation of the deformity and permits distraction and/or compression by either railed segment.

With complex upper extremity deformities, there is no standard center of rotation or arc of motion for the wrist deformity. Careful monitoring is required by the surgeon for a successful outcome. The center of rotation will change with compression/distraction, dorso-palmar or radio-ulnar correction. Proper correction and adjustments will be dependent on the age and size of the patient in addition to the specific deformity.


The Indications for the M2 are:

- Complex Upper Extremity Deformities
- Hand Contractures
- Wrist Contractures
- Indicated for Paediatric and Adult patients

1. dorso-palmar compression/distraction screw
2. dorso-palmar rail
3. screw clamp
4. dorso-palmar flange & socket
5. dorso-palmar locking screw
6. radio-ulnar locking screw
7. radio-ulnar flange & socket
8. radio-ulnar rail
9. radio-ulnar compression/distraction screw


SOFT TISSUE CORRECTION PRIOR TO CENTRALIZATION OR RADIALIZATION

1. Determine the center of rotation of the deformity. Align the M2 dorso-palmar center of rotation at the level of the capitate to serve as a visual reference for determining the first distal screw insertion point. (Figure 1)

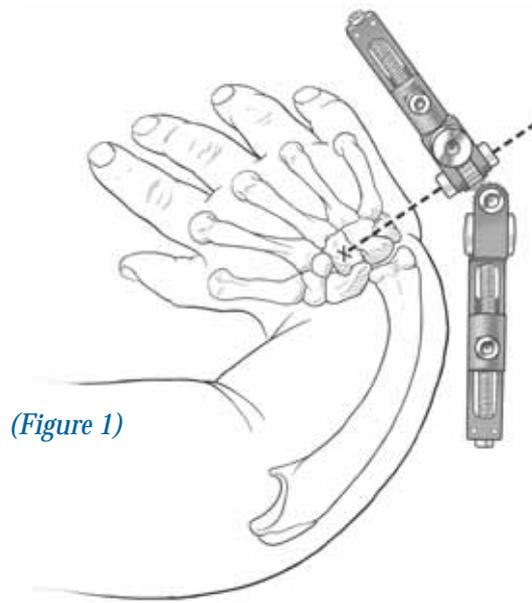


(Figure 1)

2. Position the monorails to match the deformity by using a 3 mm wrench to adjust the dorso-palmar and the radio-ulnar links. The adjustment can be refined once the rail is applied to the screws. (Figure 2)

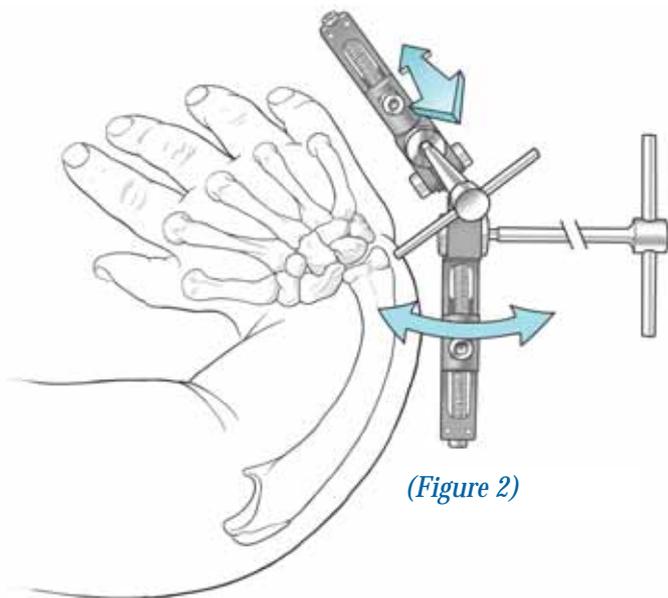


(Figure 2)

3. Ideally, allow space on either side of the screw clamps for subsequent compression and/or distraction. To move the screw clamps, turn the screw at the end of the rail with a 3 mm wrench (clockwise = distraction, counterclockwise = compression) in the end of the rail segments. In small children it may be necessary to maximize compression; i.e. position the clamp as proximal to the gears as possible on the distal rail to align the radio-ulnar axis with the gear. Although not ideal, if the 5th metacarpal is too short, placing a screw in the 4th will allow more room for the distal rail. (Figure 3)

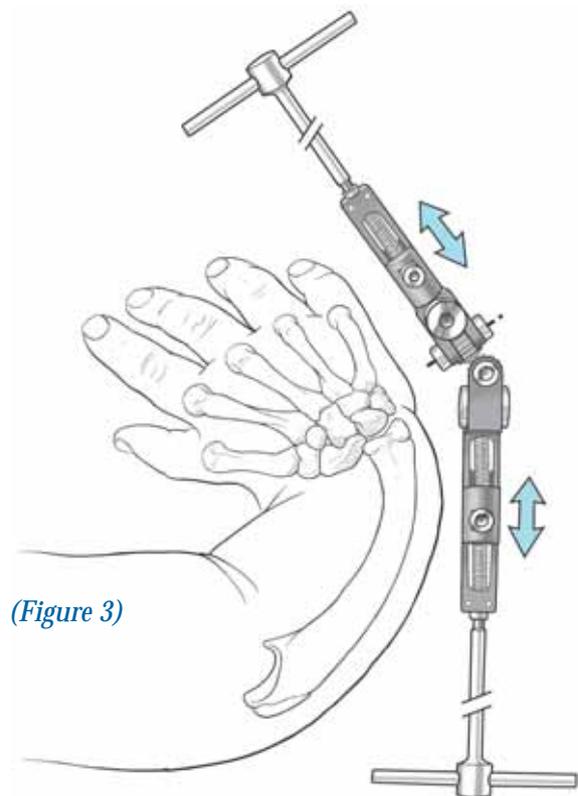

(Figure 3)

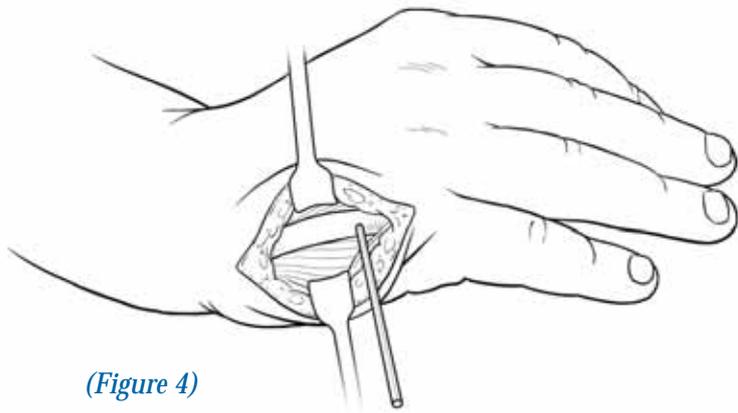

(Figure 4)

4. Make a longitudinal incision over the 5th metacarpal at the proposed position of the screws, to expose the bone subperiosteally, thereby protecting neurovascular structures. Insert the first threaded wire or screw in the frontal plane of the metacarpal, at 90° to the long axis of the bone. (Figure 4)

Note: The size of screw or wire used will be dependent on patient size. See General Comments for more information.

5. Apply the M2 MiniRail over the screw in the outermost seat of the clamp. Insert the second screw, using the clamp as a guide to ensure it is correctly spaced and parallel to the first. Only 1-2 mm of the screw tip should penetrate the second cortex in order to avoid the risk of damage to the soft tissues. Due to the tapered design, conical screws should not be advanced too far, as they will become loose if they are backed out. Once both screws are inserted, tighten the screw clamp with a 3mm wrench. (Figure 5)


Note: In small children it may be necessary to insert the screws in the two screw seats closest to the gears. A blank screw in the outermost seat is then required to ensure that the screws are all secured evenly. Where there is sufficient room, use the two outermost screw seats for greater stability.



(Figure 5)

6. Using the M2 as a guide, determine the position of the proximal screw clamp over the ulna. Make a longitudinal incision over the ulnar border, and expose the ulna subperiosteally. Insert the two proximal screws with the same procedure as for the distal screws using the outermost clamp seats, parallel to each other and in the frontal plane at 90° to the axis of the long bone. Once both screws are inserted, tighten the screw clamp with a 3mm wrench. (Figure 6)

7. Confirm accurate position of the screws with an Image Intensifier. Ensure that all screws engage both bone cortices, with 1-2 threads protruding beyond the second cortex.

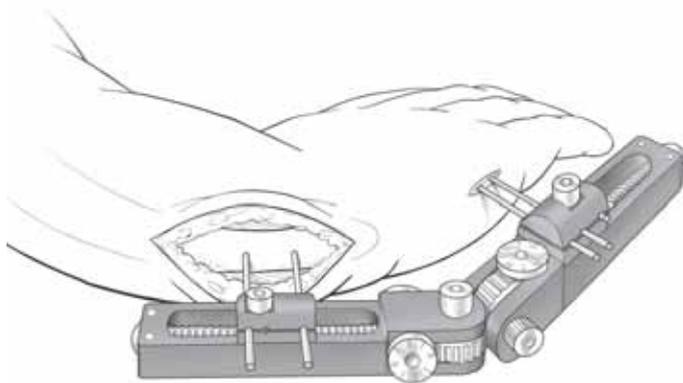


(Figure 6)


POST OPERATIVE MANAGEMENT

8. The patient should be seen in the outpatients' department on the first post-operative day. Distraction of the hand begins the first day after surgery. The family will be instructed on where and how to turn the gears on the M2 for compression/distraction, dorso-palmar and radio-ulnar corrections. The patient will be seen in the surgeon's clinic once or twice a week for the clinician to monitor the corrections and make adjustments where appropriate.

9. Distract at a rate of 1 mm per day, using the distal rail, with one quarter turn of the compression-distraction screw, clockwise four times a day. Approximately 2-5 mm of total distraction should be sufficient. (Figure 9)

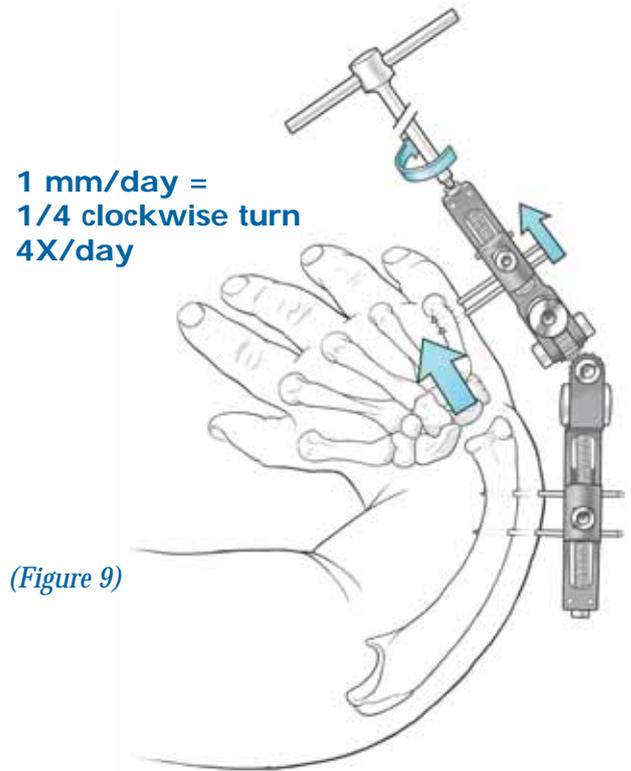


(Figure 9)

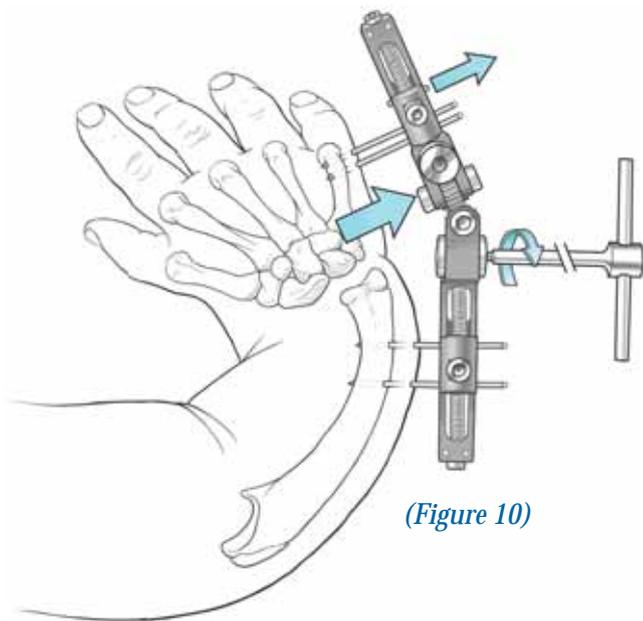


(Figure 10)

10. Correction is in an ulnar direction, 1-3° daily in 3 or 4 increments, using the radio-ulnar flange. Each mark of the flange represents 2.5°. Speed of correction depends upon the "stiffness" of the tissue and should be patient specific. During the ulnar correction, additional joint distraction of 2-4 mm or more may be required to avoid impingement. Following distraction, compression of 1-2 mm or more may be required. The axis of rotation may change as ulnar correction takes place, and lengthening may occur. It will be necessary to use compression and/or distraction to maintain the correct center of rotation. (Figure 10)

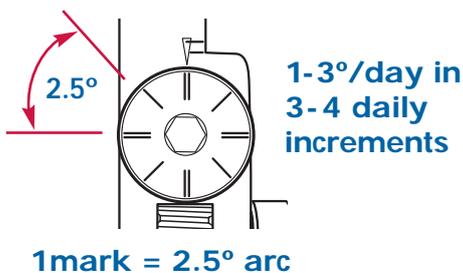


Note: With ulnar fixator placement, ulnar correction of a radial deformity will produce lengthening across the wrist. Appropriate shortening of the rails is necessary to prevent over-distraction.

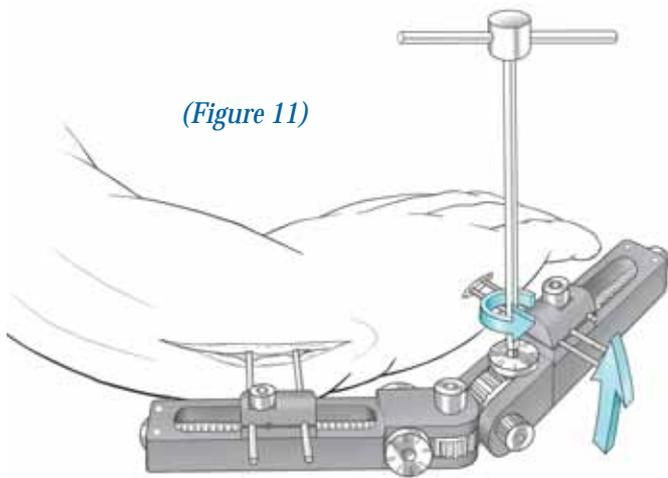

(Figure 11)


11. Dorso-palmar correction can be accomplished, if necessary, at 1° - 3° per day, incrementally, using the dorso-palmar flange. (Figure 11)

12. The fixator is applied in radial deviation to match the deformity. As correction continues in an ulnar direction, the distance between the distal ulna and proximal carpal bones will depend on the center of rotation of the fixator relative to the ulna. If the position of the center of rotation requires adjustment during correction, the rails are move proximally or distally. (Figure 12)

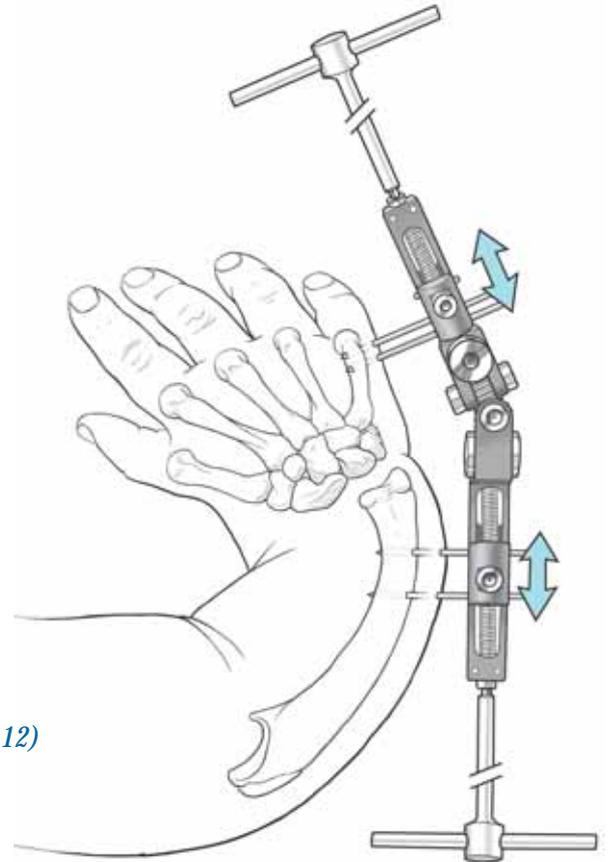

(Figure 12)

2-4 mm of over distraction

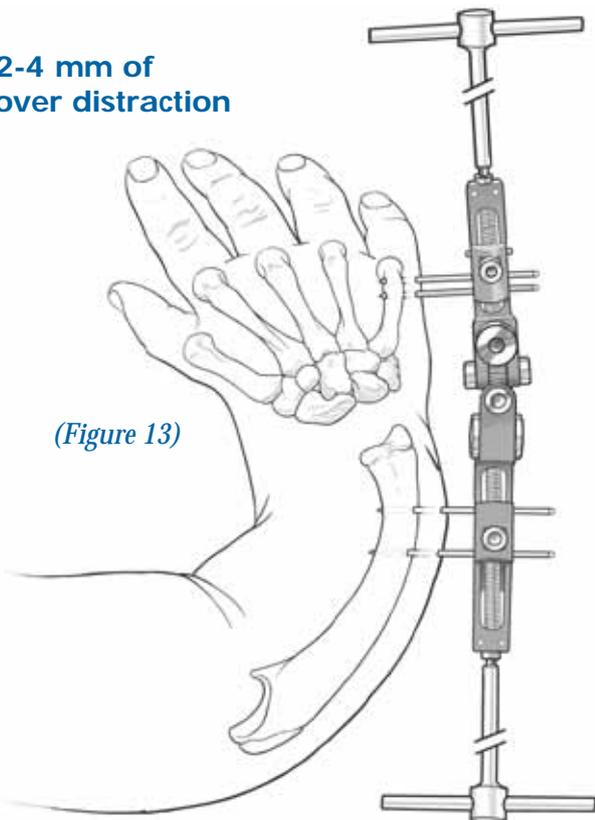


(Figure 13)

13. Continue the correction process until there is 2-4 mm of distraction at the wrist joint and radio-ulnar alignment is appropriate. (Figure 13)


14. For final alignment the hand may need to be shifted in a radial or ulnar direction; this can be achieved by loosening the distal clamp and translating the hand as required. Ulnar osteotomy may also require consideration.

CORRECTION OF WRIST CONTRACTURE SECONDARY TO SPASTICITY

1. Determine the center of rotation of the deformity. Align the M2 dorso-palmar center of rotation at the level of the radial styloid to serve as a visual reference to determine the first distal screw insertion point. (Figure 1)

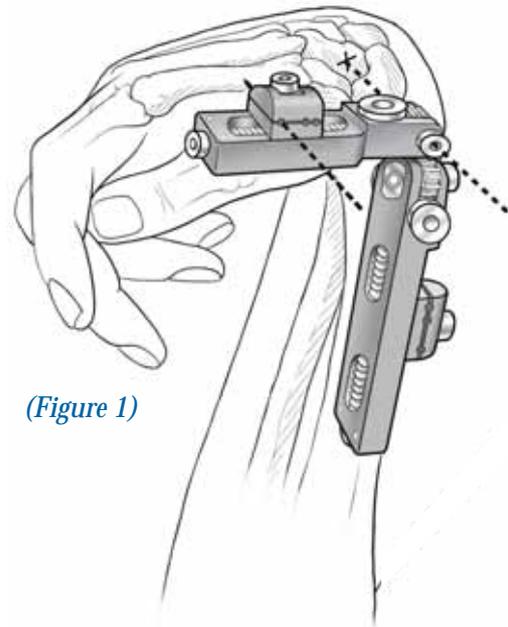


(Figure 1)

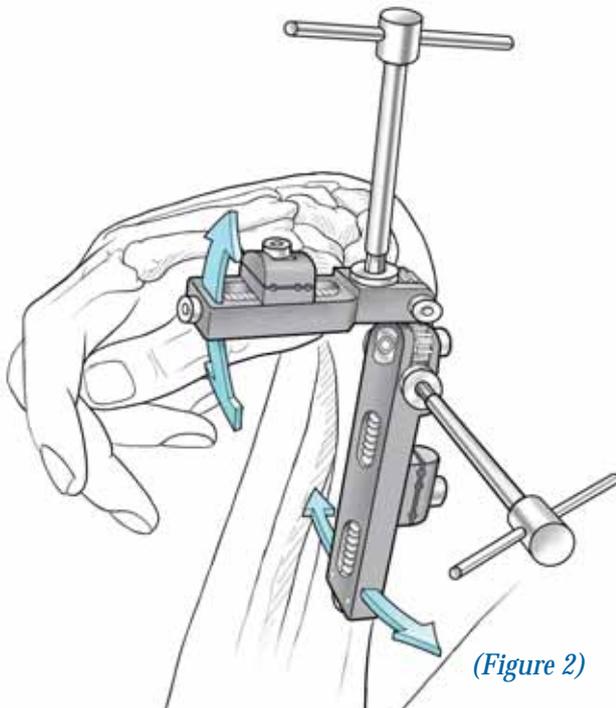


(Figure 2)

2. Manipulate the rails to match the required position of correction, using a 3 mm wrench to adjust the dorso-palmar and the radio-ulnar links. The gross adjustment can be refined after the fixator is mounted. (Figure 2)

3. Ideally, allow space on either side of the screw clamps for subsequent compression and/or distraction. Equal availability of compression and distraction in both rails is optimal. To move the screw clamps, use a 3 mm wrench in the compression/distraction screw at the end of the rail segments. (Figure 3)

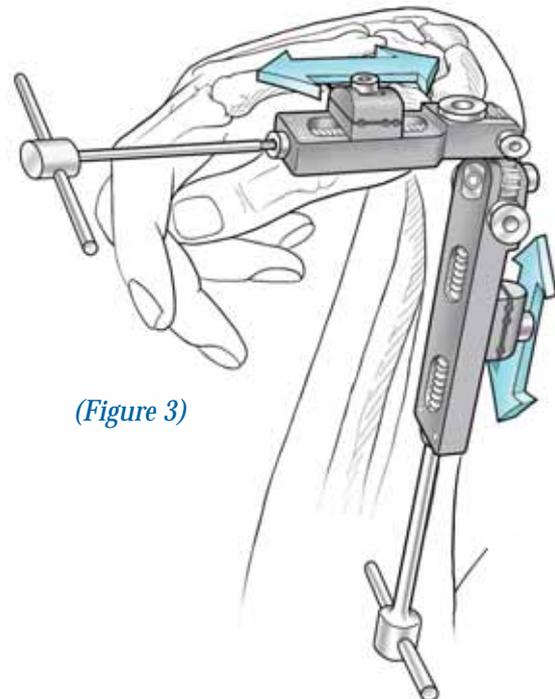


(Figure 3)

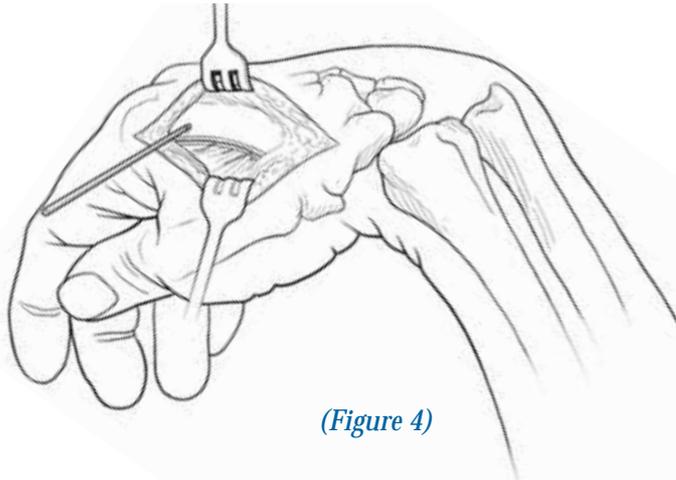


(Figure 4)

5. Apply the M2 MiniRail over the screw in an outermost seat of the clamp. Insert the second screw, using the clamp as a guide, ensuring that the screw is properly spaced and parallel to the first. The screw is positioned either in the base of the 2nd or the 2nd and 3rd metacarpals, capturing 3 or 4 cortices as necessary to obtain good bone purchase. Only 1-2 mm of the screw tip should protrude through the second cortex to avoid the risk of damage to the soft tissues. Due to the tapered design, conical screws should not be advanced too far, as they will become loose if they are backed out. With both screws inserted, the clamp locking screw is tightened with a 3mm wrench. (Figure 5)

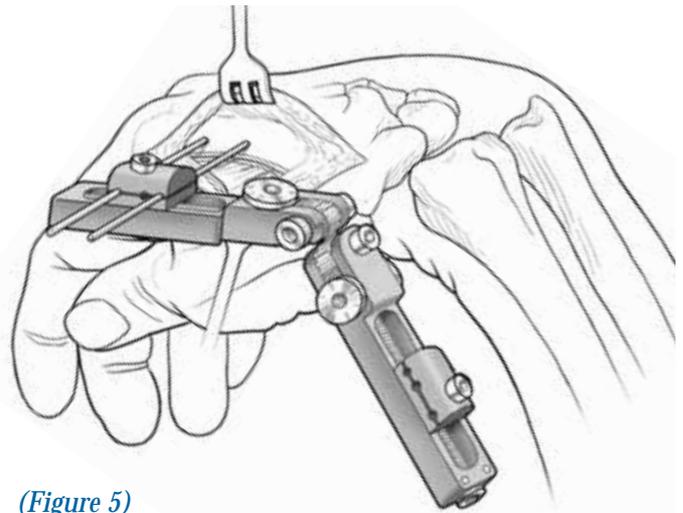


(Figure 5)

6. Using the M2 as a guide, determine the position of the proximal screw clamp over the radius. Prior to proximal screw placement, the superficial radial nerve must be identified and protected. This may be accomplished either by blunt dissection down to the bone and insertion of pin guides or by open incision, retraction of the nerve and insertion of the screw under direct vision. Insert the two proximal screws with the same procedure as for the distal screws, using the outermost clamp seats, parallel to each other, and in the frontal plane at 90° to the axis of the long bone. Once both screws are inserted, tighten the screw clamp with a 3 mm wrench. (Figure 6)

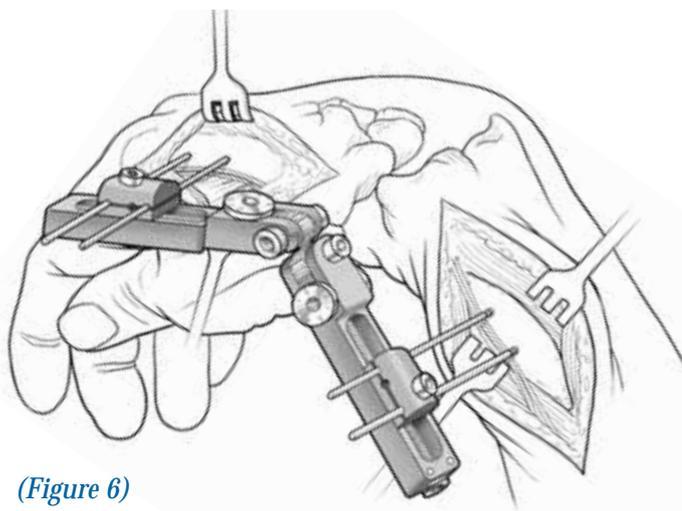

(Figure 6)

7. Confirm with an Image Intensifier that the screws are positioned accurately and that they all engage both cortices.

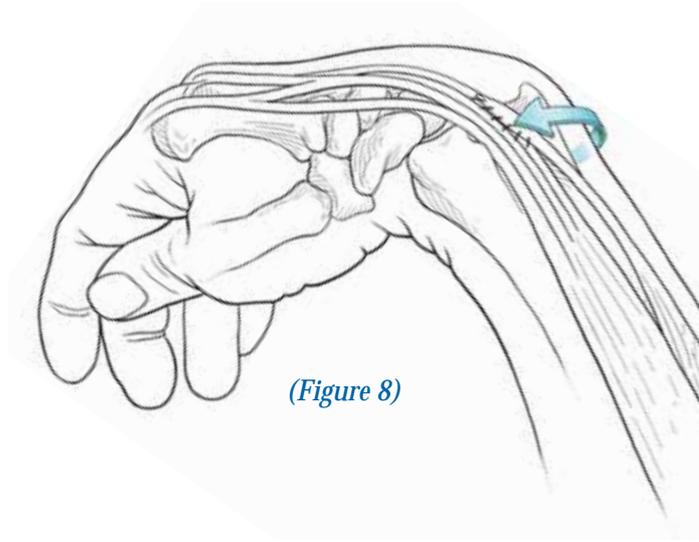


(Figure 8)

8. Tendon release is performed as necessary, and it may be easier while doing this to remove the fixator, leaving the screws in place. Options for tendon release include:

- Flexor carpi ulnaris (FCU) is lengthened or transferred to extensor digitorum communis (EDC) or extensor carpi radialis brevis (ECRB). (Figure 8)
- The flexor carpi radialis (FCR) is fractionally lengthened. (Figure 8A)
- Chemical denervation may be performed on flexor carpi radialis (FCR), flexor digitorum superficialis (FDS), flexor digitorum profundus (FDP), and flexor pollicis longus (FPL) to aid correction and decrease muscle pain.

(Figure 8A)

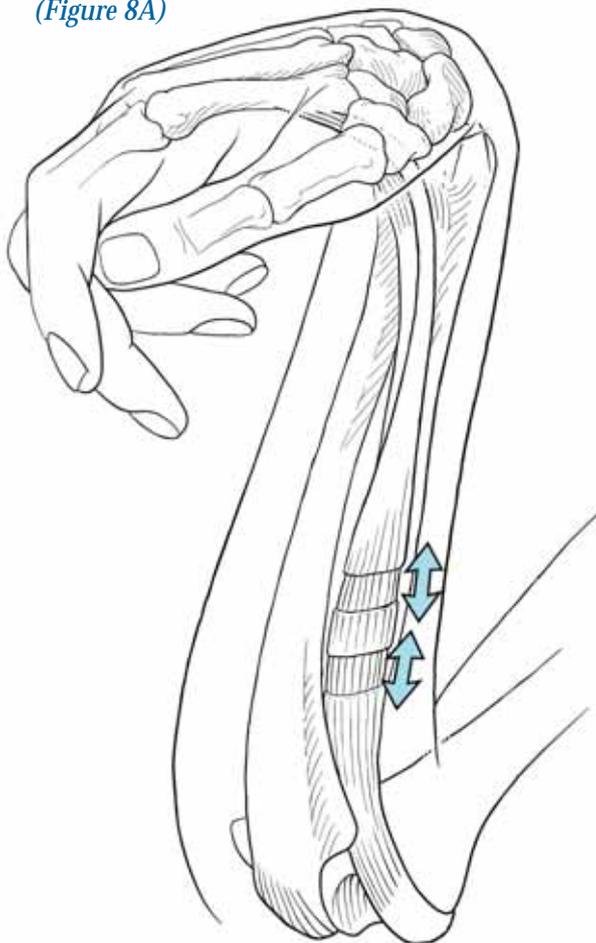


Additional staged tendon release procedures are usually required in the correction phase. The description of these procedures is beyond the scope of this manual. They should be carried out according to best practice as defined in the medical literature.

9. Proximal row carpectomy may be necessary to correct severe fixed deformity.

10. Replace the M2 over the four previously inserted screws as necessary, and tighten the clamps with a 3 mm wrench. (Figure 10)

(Figure 10)

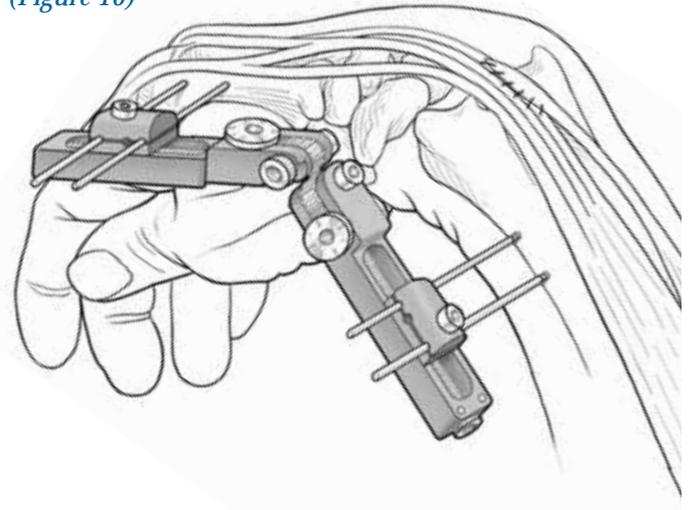


POST OPERATIVE MANAGEMENT


11. The patient is discharged and seen in clinic on the first post-operative day. The family will be instructed on where, when and how to turn the gears on the M2 for compression/distraction, dorso-palmar and radio-ulnar corrections. The patient will be seen in clinic once or twice a week for the clinician to monitor the corrections and make adjustments where appropriate. Careful monitoring is crucial. Fixator adjustments are specific to the patient and the severity of the deformity.

12. Longitudinal distraction may be required, at a rate of 1 mm per day, which is accomplished with a one quarter clockwise turn of the distal rail compression/distraction screw, four times a day, producing in all 4-5 mm of total distraction. Depending on the stiffness of the soft tissue it may be necessary to alternate distraction and then compression for the comfort of the patient. (Figure 12)

**1 mm/day =
1/4 clockwise turn
4X/day**

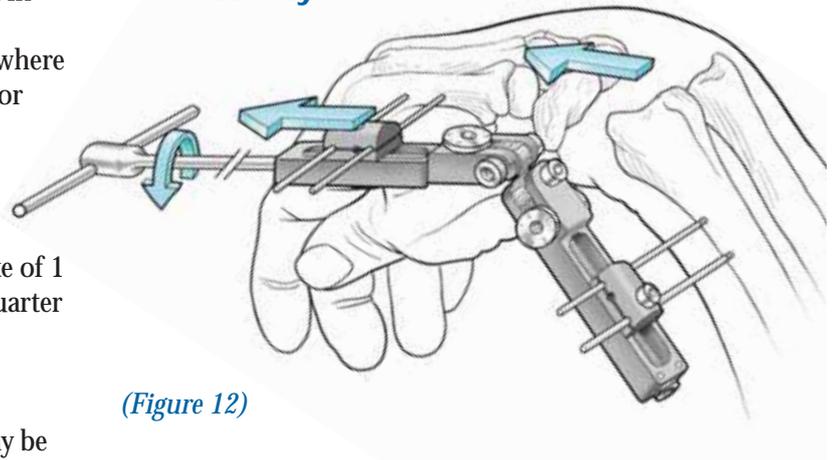

(Figure 12)

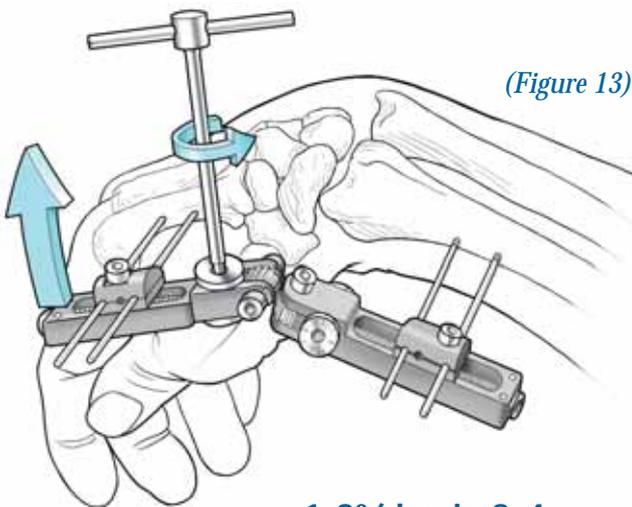

(Figure 13)

**1-3°/day in 3-4
increments daily**

13. The hand and wrist should be gradually extended at 1-3° per day in 3 to 4 increments, using the dorso-palmar flange. Passive and active motion of the digits through therapy is encouraged. (Figure 13)

Although additional staged surgeries may be required, the outcome should be improved range of dorsiflexion, grasp capabilities, release and ability to assist in daily activities.

ORDERING INFORMATION

| | |
|-------|--|
| M190 | MiniRail System Sterilization Box, Empty |
| M511 | M2 MultiPlanar MiniRail Fixator |
| 10012 | Allen Wrench 3 mm |
| 13570 | T-Wrench for Clamp Screws |
| M210 | T-Wrench for Bone Screws |
| M442 | Threaded Wire Extractor |
| M211 | Driver |
| M225 | Template Kit (4 x 1371501 Wire; 4 x 600070 Wire Guide; 4 x 600071 Pin Guide) |

Self-Drilling Cortical Screws

3.0 - 2.5 mm Thread Diameter / Shaft Diameter 3 mm

| Thread Length (mm) | | 18 | 20 | 25 | 30 |
|--------------------|-----|------|------|------|------|
| Total Length (mm) | 50 | M310 | | | |
| | 60 | | M311 | M312 | M313 |
| | 70 | | M314 | M315 | M316 |
| | 100 | | | | M317 |

Self-Drilling Cortical Screws

2.5 - 2.0 mm Thread Diameter / Shaft Diameter 3 mm

| Thread Length (mm) | | 15 | 20 |
|--------------------|----|------|------|
| Total Length (mm) | 40 | M300 | |
| | 45 | | M301 |

Threaded Wires (pack of 4)

Thread Length 15 mm / Shaft Diameter 2 mm

| Thread Diameter (mm) | | 1.6 | 2.0 |
|----------------------|-----|------|------|
| Total Length (mm) | 70 | M420 | |
| | 100 | | M426 |

Your Distributor is:

www.orthofix.com

