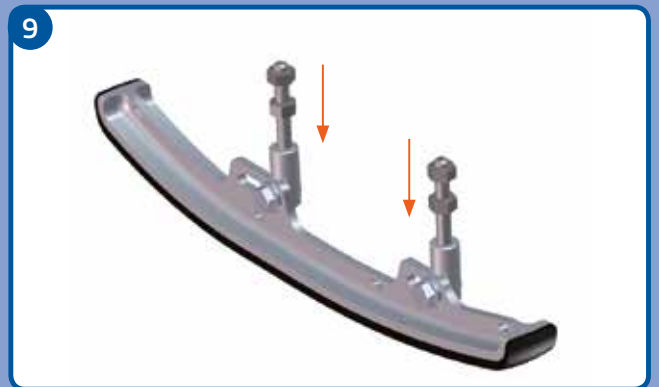
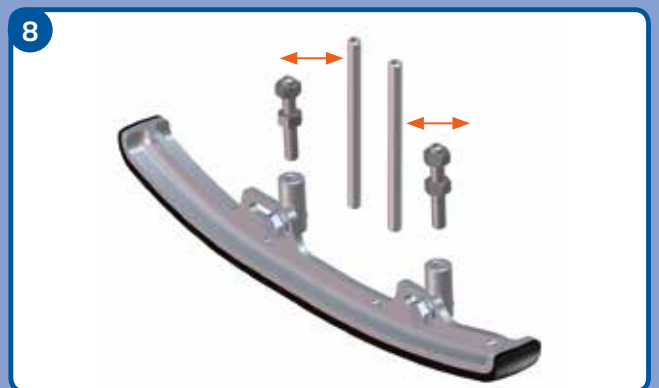
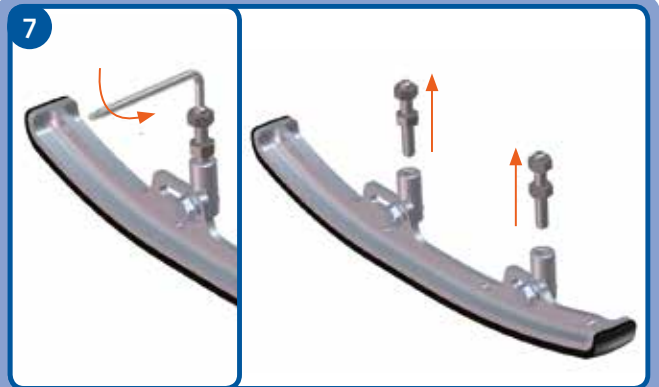
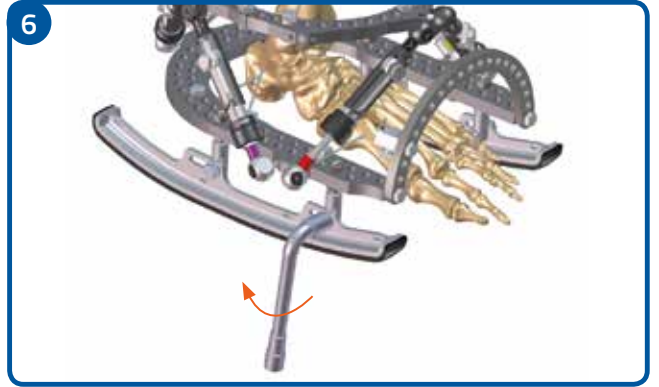
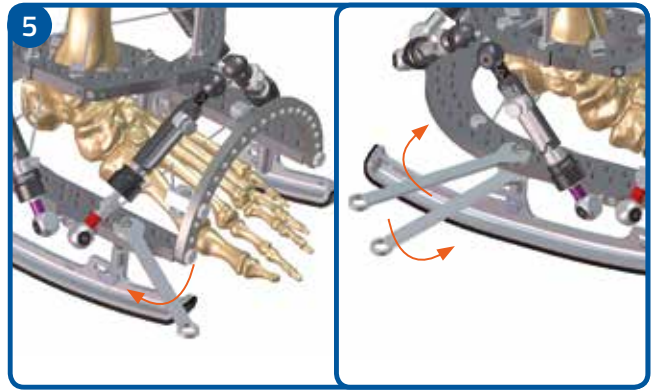
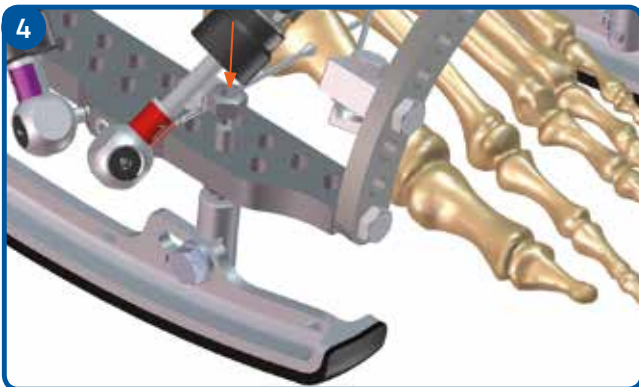
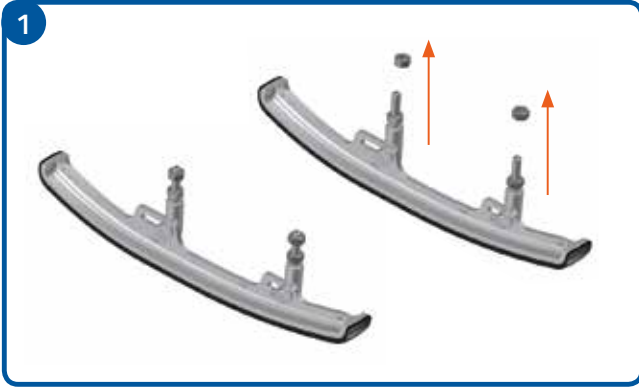
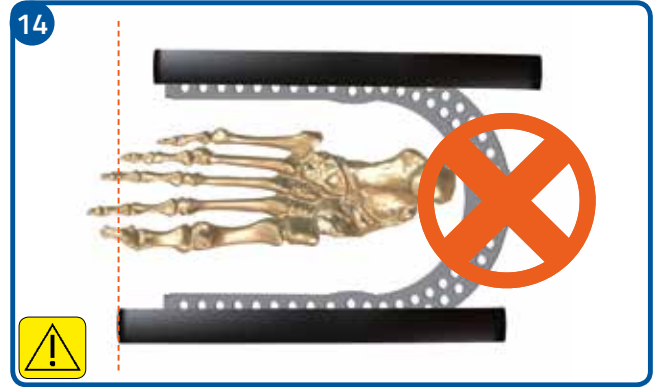
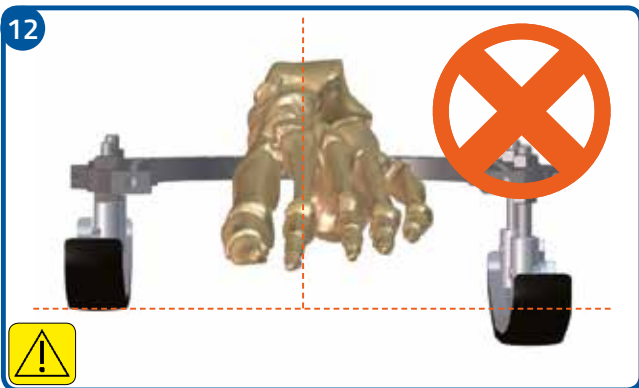
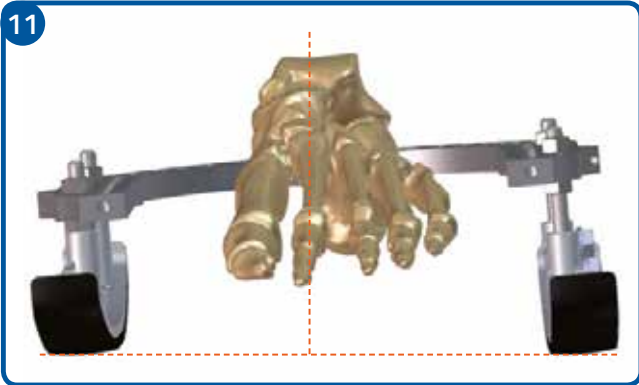
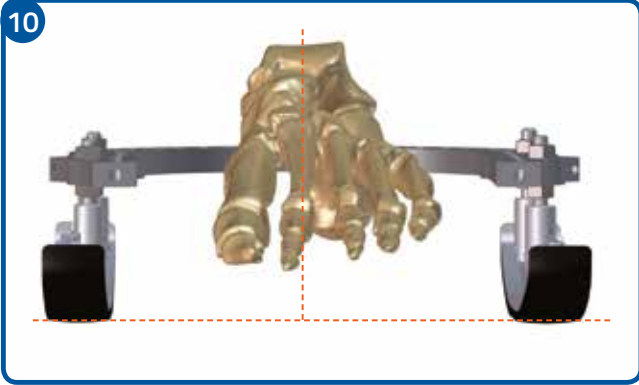
TrueLok RockerRail

56-24000

56-24000

Instruments		
	Part#	Qty.
	52-1020	1
	54-1154	2
	54-2226	1

Manufactured by:
 ORTHOFIX Srl
 Via Delle Nazioni 9, 37012 Bussolengo (Verona), Italy
 Telephone +39 045 6719000, Fax +39 045 6719380

CE
www.orthofix.com

PQ TLR1 A 09/15

ORTHOFIX®

ORTHOFIX® TrueLok RockerRail 2	EN	ORTHOFIX® TrueLok RockerRail 8	JP
ORTHOFIX® TrueLok RockerRail 2	IT	ORTHOFIX® TrueLok RockerRail 8	CN
ORTHOFIX® TrueLok RockerRail 3	FR	ORTHOFIX® TrueLok RockerRail 9	CS
ORTHOFIX® TrueLok RockerRail 3	DE	ORTHOFIX® TrueLok RockerRail 9	PL
ORTHOFIX® TrueLok RockerRail 4	ES	ORTHOFIX® TrueLok RockerRail 10	SL
ORTHOFIX® TrueLok RockerRail 4	DA	ORTHOFIX® TrueLok RockerRail 10	KR
ORTHOFIX® TrueLok RockerRail 5	FI	ORTHOFIX® TrueLok RockerRail 11	AR
ORTHOFIX® TrueLok RockerRail 5	NO	ORTHOFIX® TrueLok RockerRail 11	BG
ORTHOFIX® TrueLok RockerRail 6	NL	ORTHOFIX® TrueLok RockerRail 12	LT
ORTHOFIX® TrueLok RockerRail 6	PT	ORTHOFIX® TrueLok RockerRail 12	MS
ORTHOFIX® TrueLok RockerRail 7	SV	ORTHOFIX® TrueLok RockerRail 13	RO
ORTHOFIX® TrueLok RockerRail 7	EL	ORTHOFIX® TrueLok RockerRail 13	TR

Manufactured by:
 ORTHOFIX Srl
 Via Delle Nazioni 9, 37012 Bussolengo (Verona), Italy
 Telephone +39 045 6719000, Fax +39 045 6719380

www.orthofix.com

PQ TLR2 B 10/15

Instruction leaflet - Please read prior to use

ORTHOFIX® TrueLok RockerRail

Orthofix Srl - Via delle Nazioni 9 - 37012 Bussolengo (VR) Italy - Telephone +39 045 671 9000 - Fax +39 045 671 9380

- A. The TrueLok RockerRail is compatible with TrueLok and TL-HEX Foot Plates.
- B. Patient should be warned about hazard related to slippage.
- C. For important medical information and maintenance consult Instructions for Use PQ TLK, PQ TLH.
- D. Steam sterilization not allowed.

TrueLok RockerRail Application (Refer to pictures PQ TLR1)

1. Remove the outer locking nut from the fixation rods on the rails.
2. Loosen the adjustable sliding fixation.
3. Place the TrueLok RockerRail with the FRONT marking anteriorly onto the Foot Plate by inserting its fixation rods into the appropriate holes.
4. Place locking nuts back onto the fixation rods of the rails.
5. Use two 10mm wrenches to lock the rails onto the Foot Plate firmly.
6. The rails should be adjusted as needed for ideal placement. Lock the adjustable sliding fixation.

TrueLok RockerRail Height Adjustment

7. In the case of needing longer threaded rods, please remove the provided rods with the 1/8" Allen wrench.
8. Exchange the rods with the threaded rods of the desired length.
9. Place the provided locking nuts back on the threaded rods.

TrueLok RockerRail Final Check (Warnings & Precautions)

10. Please make sure that the two rails are placed onto the Foot Plate to provide a balanced platform to the foot, keeping it in a neutral position.
11. The rails can be placed unevenly relative to each other but, as shown in the image, they should keep the foot in a neutral position.
12. The rails should never be placed in a way to pronate or supinate the foot in static position.
13. The front and the rear of the rails should always be in line with each other.
14. The rails should never be placed offset to each other.

Instruments

	Part#	Description	Qty.
	52-1020	TL-HEX Driver 90°	1
	54-1154	Wrench, Combo 10mm	2
	54-2226	Tubular Wrench	1

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